

Monthly AED Inspection – Lifepak CR2 AED

All AEDs must be inspected monthly to ensure readiness for emergency use and for state compliance

Steps for inspecting Lifepak CR2 AEDs:

1 Check the AED cabinet and carrying case

- Are the cabinet and AED carrying case in good condition (not damaged)?
- Is the cabinet visible and not obstructed?
- Does the cabinet door open and close easily?
- Does the cabinet alarm sound when the door is open?

Readiness light should be blinking green



2 Check the AED

- Does the voice prompt come on when the cover is opened?
- Is the green readiness light blinking?
- Is the pad cover with red handle in place? [Do not pull the handle unless the pads need to be placed on a patient.]
- Is there a spare set of pads in the back of the AED case?
- Are the expiration dates good for both sets of pads?



Chest pads expiration date in corner of red handle



Spare pads in back of AED case



Rescue kit pouches:

- CPR masks (adult, pediatric)
- Non-latex gloves
- Shears and razor
- Gauze pad and wipes



3 Check the Rescue Kit, Record the Inspection

- Is the rescue kit pouch clipped to the AED case?
- Does the pouch contain: CPR mask, gloves, razor, shears, gauze pad, and wipes?
- Is the pediatric CPR mask pouch clipped to the AED case?
- Is the [monthly inspection form](#) in the cabinet or case?

Rev. 08/17/2023

If any problems are found, contact Peter Park at Peter_Park@mcpsmd.org or 240-740-7752

