

AT Tech Tip: Recording Messages on the Mini-MessageMate



InterACT Team

The Interdisciplinary Augmentative Communication & Technology Team



Mini-MessageMate

1. Turn the device on using the volume control dial on the top of the device.
2. Position the black switch next to the volume control dial to the “record unlock” position. The switch should be pushed towards the front of the device.
3. Press the blue “rec” button on the front of the device and a desired cell to record on simultaneously.
4. Speak your message **while continuing to hold** both the record button and the desired cell.
5. Release both the record button and the cell when you’re done recording your message.
6. Follow steps 3-5 to record messages for all cells.
7. Position the switch next to the volume control dial to the “record lock” position. The switch should be pushed towards the back of the device.

***Contains rechargeable batteries. Use proper adapter to recharge device.**