

AT Tech Tip:
**Recording Messages on the Big & Little
Step-by-Step Communicator**



InterACT Team
The Interdisciplinary Augmentative Communication & Technology Team



Big & Little Step-by-Step Communicator

1. Turn the switch on using the volume control knob located on the back of the device.
2. Press the record button (located on the back of the switch) until the LED light turns red.
3. Release the record button once the LED light is on.
4. Press and **hold down** the colored switch surface while you speak your message.
5. Release the colored switch surface when you're done recording your first message.
6. Press the colored switch surface again and **hold down** while you speak your next message and then release. Continue this step for as many messages as you wish to record.
7. Once all messages have been recorded, press the "Record" button once (the LED light will turn off).

***Requires one 9-volt battery.**