

AT Tech Tip: Recording Messages on the BIGmack Communicator



InterACT Team

The Interdisciplinary Augmentative Communication & Technology Team



BIGmack Communicator

1. Turn the switch on using the volume control dial located on the side of the switch.
2. Press and **hold** the record button (also located on the side of the device).
3. While pressing the record button, simultaneously **press and hold** the large, colored switch surface.
4. While holding both the record button and the colored switch surface, speak your message.
5. Release the record button and the colored switch surface when you're done recording your message.

***Requires one 9-volt battery.**