

AT Tech Tip: Recording Messages on the Step by Step Communicator 75



InterACT Team

The Interdisciplinary Augmentative Communication & Technology Team



Step-by-Step Communicator 75

1. Turn the switch on using the volume control dial on the back of the switch.
2. Press the “repeat/rec” button on the back of the device **twice** and release.
3. Press the colored switch surface and **hold down** while you speak your message.
4. Release the colored switch surface when you’re done recording your first message.
5. Press the colored switch surface again and **hold down** while you speak your next message and then release. Continue this step for as many messages as you wish to record.
6. Once all messages have been recorded, press the “repeat/rec” button once.

***Requires one 9-volt battery.**