

## **AT Tech Tip: Recording Messages on the Voice-In-The-Box (16 locations)**



### **InterACT Team**

*The Interdisciplinary Augmentative Communication & Technology Team*



### **Voice-in-the-Box (16-locations)**

1. Open the gray flip door on the back of the device.
2. Use the black knob to select the level you wish to record onto.
3. Press the red record button and a cell on the device simultaneously.
4. Speak your message **while continuing to hold** the record button and the desired cell.
5. Release the record button and the cell when you're done recording your message.
6. Follow steps 3-5 to record messages on all cells for the desired level.
7. Follow steps 2-5 to record messages for another level.
8. Close the gray flip door.

**\*Requires 4 AA batteries.**